
Calcified Shoulder (calcified tendonitis)

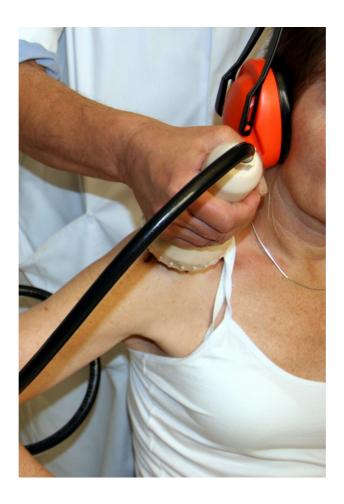

This is the musculoskeletal disorder most frequently treated with CellSonic VIPP. It is usually the result of excessive stress and strain. The doctor will locate the calcific deposits by aid of ultrasound scans.

A local anaesthetic such as lignocaine should be used. Use a spark head of the CellSonic machine with a focal length of 20 mm and apply 1,200 pulses. This takes six minutes. An improvement can be felt after several days and within 4 to 6 weeks all could be cured.

If not, a second treatment the same as before should resorb the remaining calcific deposits. Space the treatments about a month to 6 weeks apart. Only occasionally is a third treatment needed.